




## EpiPen®

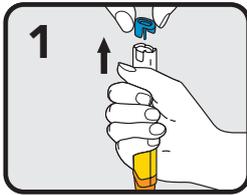

1 Form fist around EpiPen® and pull off blue safety release

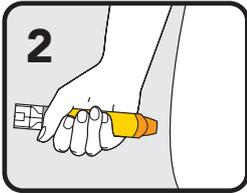


2 Hold leg still and place orange end against outer mid-thigh (with or without clothing)

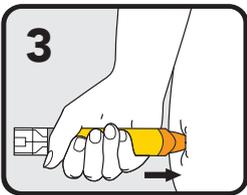

3 Push down hard until a click is heard or felt and hold in place for 3 seconds. Remove EpiPen®

EpiPen® Jr (150 mcg) is prescribed for children 7.5 - 20kg  
EpiPen® (300 mcg) is prescribed for children over 20kg and adults


## Jext®

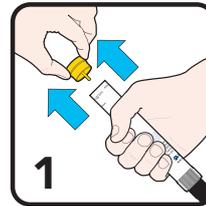

1 Form fist around Jext® and pull off yellow cap



2 Place black injector tip against outer mid-thigh (with or without clothing)

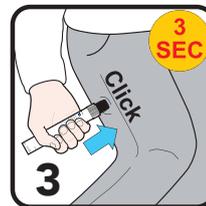


3 Push black tip firmly until a click is heard and hold in place for 3 seconds. Remove Jext®

Jext® Jr (150 mcg) is prescribed for children 7.5 - 20kg  
Jext® (300 mcg) is prescribed for children over 20kg and adults


## Anapen®



1 Pull off black needle shield

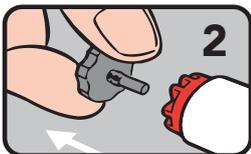

2 Pull off grey safety cap from red button

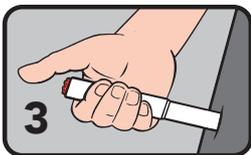


3 Place needle end firmly against outer mid-thigh at 90° angle (with or without clothing)




4 Press red button so it clicks and hold for 3 seconds. Remove Anapen®

Anapen® 500 is prescribed for children over 50kg and adults

## neffy®

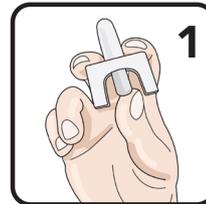

1 Hold as shown. Do not test spray

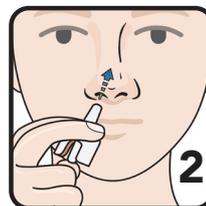

2 Place nozzle into nostril until fingers touch nose.



3 Press plunger firmly

neffy® 1mg is prescribed for children 15 - 30kg (4 years and over)  
neffy® 2mg is prescribed for children and adults 30kg and over

Follow the ASCIA Action Plan or First Aid Plan for Anaphylaxis

Each device is for single-use only.

Give the used device to the ambulance with the time it was given.



Scan this QR code for instructions on the ASCIA website.